

Product Catalog Clinical Range 2019



Experience The Refinement

Shiva Products

ISO 9001 : 2015 & 13485 : 2016

Proudly Exporting to 40+ Countries Worldwide

INDEX

NEW LAUNCH

CeraSeal-B.....	5
MTA.....	6
Twin Kleen.....	7
Easy Tray.....	8
Hemostaz Solution.....	9

ENDO RANGE

Sealmax-R.....	10
Calx.....	10
Calx-O.....	11
Calahyd-RC.....	11
Endo Paste.....	12
Gsol.....	12
Chloro-X.....	13
Sodium Hypochlorite.....	13
F-Sol.....	14
D-Vitz.....	14
Endo Gel & Endo-L.....	15
T-Fill.....	15

PREVENTIVE RANGE

Find Fast.....	16
Zirprophy.....	16
Pro-Paste.....	17
Plaque-D.....	16

PROSTHO RANGE

Mock-up wax.....	18
Bite Registration Wax.....	18
Green Tracing Sticks.....	19
Shellac Base Plates.....	19
Impression Composition.....	20
Special Tray Plates.....	20
Modelling Wax.....	21
Inlay Sticks.....	21
Casting Wax Sheets.....	21
Die Stone Plaster (Pink).....	22
Reinforcement Mesh.....	22
Cold Cure Powder / Liquid.....	23
Cold Mould Seal.....	23
Heat Cure Powder / Liquid.....	24
Sticky Wax Sticks.....	24

RESTORATIVE RANGE

ETC-37.....	25
Eugenol.....	25
Calcium Hydroxide Powder.....	26
Mercury.....	26
Zinc Oxide.....	26

OTHER

Lubri Spray.....	27
Mouth Mirrors.....	27
Arch Bar.....	28
Glass Beads.....	28
Relief Wax for Ortho.....	28

COMPANY PROFILE

An ISO 9001 : 2015 & 13485 : 2016 accredited company catering a wide range of products such as dental material, dental products, dental alloy, jewellery wax, etc...

An Overview

Shiva Products, established in the year 1998, an eminent company is a manufacturer for Dental Products and other specialty waxes. With more than two decades of experience in this industry, we are proud to be recognized as a renowned manufacturer and exporter of premium quality Dental Goods, Jewellery Waxes, Investment Casting Waxes, Modelling Waxes, Dental Alloys, Dental Instruments, etc., which are specifically made according to international quality parameters. We are a team of experienced professionals working with a common goal of quality production, thereby, winning the trust and faith of thousands of national and international clients.

The Company has ventured into the field of dentistry since 1998 with 2 products in hand. With Team work & focus on **RRR strategy (Reliable Responsible & Reasonable)**, we currently have more than 80 Products catalogued and many in pipe line. Company has become one of the leading manufacturer in Domestic & well-known in International Market. The Strong relationship and the trust of consumers, trade and the dental profession built over two decade of operations in India has made 'MAARC' a trusted Brand.

'MAARC' products have the repo of "**RIGHT QUALITY RIGHT PRICE**" in India as well as International market. Our An ISO 9001 : 2015 & 13485 : 2016 manufacturing standards helps us to deliver quality dentistry products at affordable rates to masses. Our goal is to build confident relationship with our customers.

We also cater to manufacturers in neutral packing and contract manufacturing can also be done.

DENTAL SEGMENT:

Clientele:

01. Domestic: We are currently selling to approx. 1000-1200 Distributor / Dealers (many states of India), approx. 500-600 Laboratories within Mumbai, 4000-5000 dentists are been catered directly / through distributor/dealer network through our company within Mumbai + Institutional supply + government Tender supply.

02. Exports - Currently supplying to approx. 40 countries across various continents

Certifications: Shiva Products has several certifications as under.

- a) EN ISO 9001: 2015
- b) EN ISO 13485 : 2016
- c) Class I device products are all CE certified
- d) Class IIA device applied for Product Certification with TUV
- e) EAR Obelis SA - Belgium
- f) Registration with various Chamber of commerce in Mumbai for smooth functioning of export orders.

COMPANY PROFILE

Vision

We are led by global vision and drive to manufacture various dental material comparable to international standards so as to offer optimal customer satisfaction.

With a goal to establish strong relationships with more and more customers, we offer our Products at extremely affordable price. It is also a continuous effort on our part to deliver the orders within the stipulated time.

Assured Quality

We take utmost care of quality while producing our goods, which meet international quality norms in all respects. Shiva Products has been certified with an ISO 9001 : 2015 & 13485 : 2016 certification, which is a testimony in itself to our quality production. Each operation is executed under the strict supervision of trained professionals in accordance with standardized production norms and methods, thus leaving no stone unturned in ensuring premium quality.

Infrastructure

Shiva Products has a state-of-the-art manufacturing unit, where latest manufacturing machines and equipment contribute substantially in manufacturing global quality dental products. Our manpower includes engineers, dentists, technicians, designers, R&D team, QC, marketing professionals, quality personnel in addition to some semi -skilled workers. We work as a team to achieve our vision.

In-process Quality Analysis: Our manufacturing unit has its own lab and checking facilities wherein quality analysis is been done on daily basis for RM, in-process and final product before its sent to the packing zone.

The most advantageous and beneficial technique of production supplied by our R&D team give us an edge over our competitors. Our warehouse is a boon, as it facilitates safe storage of bulk products to meet all types of market demands.

Future Plans

Our Company want to take advantage of the booming export market to add to its market share and not lose the opportunity to get an insight into the behavior of western markets and its consumers. Led by a global vision and a drive to maintain quality upto international standards, our company strives to produce products conforming to international standards.

We would like more distributors to ADD their companies to our network and build beautiful Smiles all over!!

New Launch

CeraSeal-B

Packing	Product Code
1 gm powder, 5 ml liquid, measuring spoon and mixing pad	8501/001



- Excellent Handling
- Hermetic seal
- Radiopaque
- Suitable as lining material
- Pure Mineral Tech :No Discoloration
- Bactericidal
- Biocompatible
- Good radiopacity

New Launch

MTA

Packing	Product Code
1 gm powder, 5 ml liquid, measuring spoon and mixing pad	8601/001



- Excellent Handling Properties
- Fine Grain
- Fast Setting
- Pure Mineral Tech : No Discoloration
- Bactericidal effect due to High pH
- Biocompatible
- Promotes mineralization of tissue
- High Strength make is suitable as base restoration material
- Good radiopacity

New Launch

Twin Kleen

Packing	Product Code
100 Capsules	3014/100

**The All in One
Irrigating Solution**

**Just Mix Powder with
Sodium Hypo**



Eazy Tray

Packing	Product Code
100 gms	9701/100

New Launch

- For Lucia Jig
- Bite Stop
- Special Tray




New Launch

Hemostaz Solution

Packing	Product Code
15ml	3016/015
3gm Syringe with 10 Pcs Applicator tips	3015/003

- Stop Bleeding in Seconds, and Decreases the Need to retake Impression
- Hemostatic Agent for Crown & Bridge Procedures
- Viscous - Stay in Place when Delivered
- Unique Binding & Coating Agents

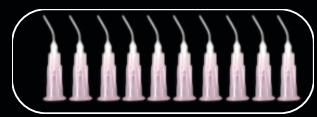

Sealmax – R



Known as : Resin based root canal sealer

Packing	Product Code
10 gm powder, 10 ml liquid, measuring spoon and mixing pad	8001/010

Description:

▶▶ Sealmax-R contains fine particles to ensure proper sealing

Shelf life and storage: 30 months under proper storage. Always close the cap immediately after use.

Properties & Advantages

- ▶▶ Eugenol Free
- ▶▶ Insoluble in water and non degradable
- ▶▶ Minimum shrinkage at room temperature

Benefits :

- ▶▶ Grain Free
- ▶▶ Hydrophobic
- ▶▶ Bacteriostatic
- ▶▶ Negligible Shrinkage
- ▶▶ Complete Impervious Seal
- ▶▶ Excellent Adhesion
- ▶▶ Tissue Tolerant
- ▶▶ Most Ideal Viscosity
- ▶▶ Best Intracanal Sealing
- ▶▶ Excellent Radio Opacity

Calx

Known as : Calcium Hydroxide Paste (Regular)

Packing	Product Code
3gm Syringe with 3 Pcs (18 G) + 2 Pcs (26 G) Applicator tips	9101/003

Description :

▶▶ It is premixed high pH Calcium Hydroxide Water based paste with stabilizer

Indication :

- ▶▶ Flowing Canal
- ▶▶ Hard Tissue Formation
- ▶▶ Root Resorption
- ▶▶ Radiolucent

Properties & Advantages

- ▶▶ Ready to use, radiolucent, premixed, non separating Calcium Hydroxide Paste which is useful in periapical infection
- ▶▶ Used as a liner.
- ▶▶ It reduces the hypersensitivity of the dentine and helps the formation of secondary dentine.
- ▶▶ Non Separating Homogenous Mix
- ▶▶ Ideal Viscosity & Micronised Calcium Hydroxide for Effective Application
- ▶▶ Easy Insertion in Canal Thanks to "MAARC ADHERE TECH"

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use.



Calx-O



Known as : Water Based Calcium Hydroxide Paste (With Barium Sulphate)

Packing	Product Code
3gm Syringe with 3 Applicator tips	9102/003


Indications :

- ▶▶ Flowing Canal
- ▶▶ Apexification - Hard tissue barriers
- ▶▶ Root Resorption

Advantages : Radio-opaque with Excellent bio-compatibility with no toxic effect on cells. Easy handling and easy to apply. Ideal for infected root canal and vital pulpotomies in deciduous teeth. It can also be used in conjunctions with gutta percha points and regular root canal sealants.

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use.

Calahyd-RC

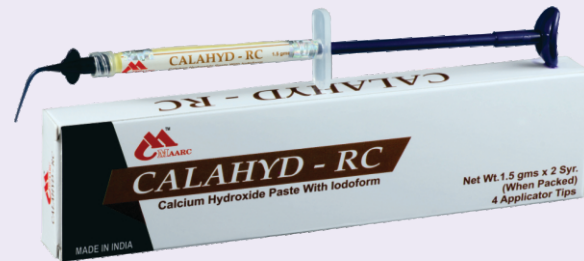

Known as : Calcium Hydroxide Paste With Iodoform

Calahyd - RC is premixed Calcium Hydroxide Paste containing Iodoform. The addition of iodoform to calcium hydroxide improves radiopacity and adds an antibacterial agent to the paste. Non hardening paste, solublized calcium hydroxide remains active in root canal for a longer time.

Benefits :

- ▶▶ Non-Separating Homogenous Mix
- ▶▶ Bacteriostatic & Antibacterial Properties
- ▶▶ Effective in Abscess
- ▶▶ Promotes Apexification
- ▶▶ Easy Retrieval from the Canal Using ENDO-L (EDTA LIQUID)+10% Diluted Citric Acid Solution

Applications : For pulp capping and pulpotomy. Apexifications and hard tissue formation. Temporary sealing material for infected root canals.

Method of Use : After root canal preparation, clean and dry root canal. Attach disposable needle to the Syringe and check the flow of paste through the needle. Insert needle into the canal, about 2mm short of the apex. Fill canal completely by pushing the plunger while slowly withdrawing the syringe. Clean excess paste with a sterile cotton pellet.

Packing	Product Code
1.5 x 2 Syringes with 4 Applicator tips	9103/15X2

Storage & Shelf Life : Best before 24 months keep away from direct flame. Store in a cool and dry environment. Always close the cap immediately after use. Do not freeze.

Endo Paste



Description

It is water soluble Ethylene Diamine Tetra Acid (EDTA) Paste for root canal cleaning

Packing	Product Code
3gm Syringe with 3 Applicator tips	9203/003

Indication of Use

- ▶▶ Chelating (Calcium Binding) agent
- ▶▶ For chemical and mechanical cleaning of the root canal
- ▶▶ Shaping of root canal
- ▶▶ To be used with sodium hypochlorite. Don't use on soft tissues light skin.

Advantage :

Contain 3 lubricant for easy reaming root canal

Shelf Life : 24 months from date of manufacturing. Always close the cap immediately after use.

Benefits :

- ▶▶ **GLIDE YOUR FILES:** With Unmatched Lubrication
- ▶▶ **EASYCOAT:** Coats File With Ease
- ▶▶ **REMOVE SMEAR LAYER EFFECTIVELY**
- ▶▶ **EFFERVESCENCE:** Reacts With Sodium Hypo To Flush Debris
- ▶▶ **HYDROPHILIC:** Easy Wash Off
- ▶▶ **NO WASTAGE:** Better Stability Non Drying & No Phase Separation

Known as : Gutta Percha Solvent D-limonene based G. P solvent for removing Gutta Percha at the time of root canal re-treatment

Packing	Product Code
15ml	3006/015

Indications:

- ▶▶ To remove all types of root canal filling materials including Zinc-Oxide-Eugenol cement
- ▶▶ Fitting of the primary gutta percha point in root canal filling techniques using the lateral condensation method.
- ▶▶ Retreatment after root canal filling by gutta percha condensation.
- ▶▶ For removing gutta percha at the time of root canal re-treatment

Benefits :

- ▶▶ G-Sol instantly softens GP (**Rapid Action**)
- ▶▶ Easy removal of Sealer once GP soften
- ▶▶ Orange oil based (**Pleasant flavour**)

Product Benefits :

Non hazardous/non carcinogenic unlike chloroform/trichloroethylene	Less volatile as compared to chloroform / trichloroethylene
Based on natural substances d-limonene, doesn't cause irritation to oral mucosa and periapical tissue	Evaporation of solvent is less due to low volatility

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use.

Gsol



Chloro-HX - 2%



2% Chlorhexidine Solution that is used to clean & disinfect before many dental procedures. This greatly reduces the potential for post-operative sensitivity caused by residual Bacteria.

Packing	Product Code
100ml	3010/100

Shelf Life : 24 months from the date of Manufacturing.


Sodium Hypochlorite

Packing	Product Code
(3%) 100ml	3009/100
(5%) 100ml	3004/100
(3%) 250ml	3009/250
(5%) 250ml	3004/250

Properties & advantages :

- ▶ Used as endodontic irrigant, debridement and disinfection of root canals during and after instrumentation

Shelf life and storage: 24 months under proper storage.

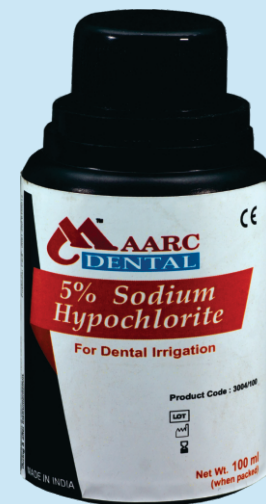

F-sol



Known as : Formocresol Solution

Packing	Product Code
15 ml	3012/015

INDICATIONS

Indicated as a disinfectant and devitalizing solution used as part of endodontic treatment.

STORAGE

Store between 59°-86°F (15°-30°C). Protect from freezing.

Shelf life and storage : Best before 24 months under proper storage.

D-Vitz (Pulp Devitalizer)



It is a pulp devitalizing Paste which provides powerful pulp devitalizing properties because of paraformaldehyde & lidocaine. It is Arsenic free paste

Packing	Product Code
3 gms syringe + 5 Applicator tips	9003/003
10 gms	9003/010

Shelf life and storage: 24 months under proper storage.



Endo Gel

Known as : 17% EDTA Gel

Packing	Product Code
3 gms syringe with 3 Applicator tips	9202/003
100 ml.	9201/100

Properties & Advantages :

- ▶▶ Effective chelating, lubrication, deburring water soluble material.
- ▶▶ It facilitates root canal preparation.

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use.

Benefits :

- ▶▶ 17% Liquid based EDTA
- ▶▶ Used to flush out the debris from canal
- ▶▶ Removes the smear layer from dentinal walls
- ▶▶ Also helps in removal of calcium hydroxide oil based from the canals
- ▶▶ Mechanically Activated using a endo brush



Product Description :

T-Fill is ready to use temporary filling material self-curing under humidity for temporary filling of cavities.

Packing	Product Code
40gm pack, mint flavour, colour white	5507/040

Indication of use :

- ▶▶ For use as Temporary filling material after Endodontic treatment
- ▶▶ For use as Temporary Inlay material

Properties :

- ▶▶ Eugenol Free offer non irritating properties.
- ▶▶ Higher adhesion to teeth structure.
- ▶▶ Micronised and homogeneously mixed for grain free consistency. Gives best adaptability.
- ▶▶ Quick setting in moist conditions
- ▶▶ There is minor but even expansion on setting so no cracks develop, giving best sealing hence Protecting to Pulp.

Storage : After opening store in refrigerator (don't Freeze) .

Shelf life and storage : 24 months under proper storage.

T-Fill



Benefits :

- ▶▶ Excellent Adaptability
- ▶▶ Non Sticky to Instrument
- ▶▶ No Cracks: Even Expansion Due to Even Grain Size
- ▶▶ Eugenol Free
- ▶▶ Pleasant Flavor : Mint
- ▶▶ Neutral Taste
- ▶▶ Minimum Moisture Contamination: Dual Cap For Air Sealing

Find Fast



Known as : Caries Indicator

Packing	Product Code
15ml	3005/015

Properties & Advantages :

- ▶▶ Find Fast is made from special dye
- ▶▶ It stains protein ensures uninfected areas is not stained
- ▶▶ Identifies carious dentin so you can ensure the maximum amount of inner dentin is preserved and infected dentin is removed
- ▶▶ Identifies carious dentin ensures the maximum amount of inner dentin is preserved and infected dentin is removed.
- ▶▶ Easily locate root canal orifices and calcified canals.

Shelf life & storage: 24 months under proper storage.

Zirprophy CE

Known as : Prophylaxis Paste

Contains : Zirconium Silicate.

Packing	Product Code
45 gms	9001/045

Properties & Advantages :

- ▶▶ Zirprophy contains Zirconium Silicate.
- ▶▶ It does not break down as pumice from coarse to fine but rather removes stain smoothly due to fine particles.
- ▶▶ Zirprophy is to be used light handedly to avoid damage to enamel
- ▶▶ Thixotropic Oil Free
- ▶▶ Splatter Free
- ▶▶ Fluoride Free
- ▶▶ Spearmint Flavor

Shelf life and storage : Best before 24 months under proper storage. Always close the cap immediately after use. Do not freeze.



Benefits :

- ▶▶ Zirconium silicate abrasive media
- ▶▶ Easy removal of smoking and tar tar stains
- ▶▶ Controlled particle size (Minimum post scaling sensitivity)
- ▶▶ Better polish
- ▶▶ Fluoride Free

PREVENTIVE RANGE

Pro-Paste



Known as : Prophylaxis Paste

Packing	Product Code
90 gms	9002/090

Description:

▶▶ Pro-Paste is a pumice paste for traditional polishing and stain removing use

Properties & Advantages :

- ▶▶ Thixotropic Oil Free
- ▶▶ Splatter Free
- ▶▶ Mint flavor
- ▶▶ Do not Freeze

Shelf life and storage : 24 months under proper storage.
Always close the cap immediately after use. Do not freeze.




Plaque-D

Packing	Product Code
2gm x 4 syr. + 4 applicator tips	9301/010


PLAQUE-D : Plaque Disclosing Gel is made of special dye which stains plaque, As a result, one gets clear idea of brushing irregularities. Dentist can easily make patients understand in which area they have to focus more and which technique of brushing to be use for more effective cleaning. This is one of the best way to educate patients about their brushing technique and teach them proper brushing.

Dentist can give one syringe to patient to try the same at their home.

Shelf life and storage : 24 months under proper storage.

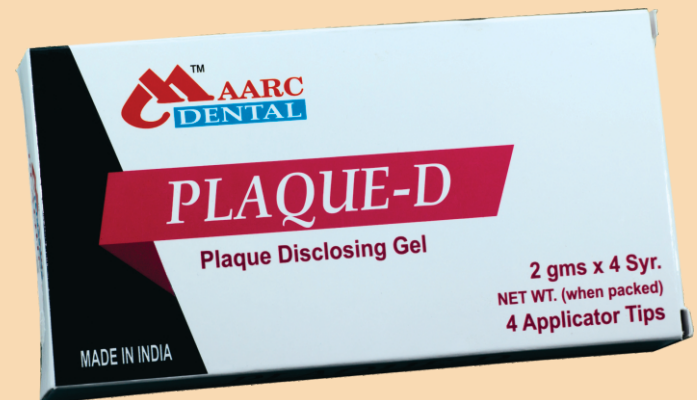

Mock-Up Wax CE

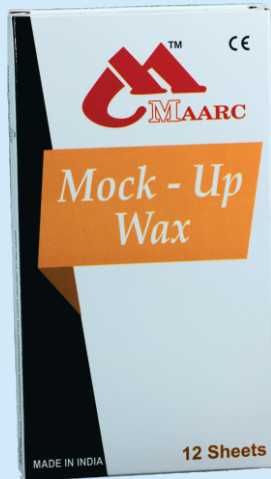

Packing	Product Code
2 Sheet	2201/225
200 gms	2201/200

Color : White

Product features :

- ▶▶ It simulates the final outcome. The result is temporary and can be changed according to the patients desire
- ▶▶ It can be repeated as many times as needed in order to select the best possible solution
- ▶▶ It helps create trust between the patient and the dentist
- ▶▶ It allows the dentist to evaluate the patient's desires and change the treatment plan accordingly

Shelf life and storage : Best before 60 months under proper storage.

Thickness : 1.3mm - 1.5mm. (Custom Size / Thickness Catered),

Size : 160mm X 90 mm

Packing : Each Packet of 'MAARC' Mockup Wax contains 12 sheets wrapped in non adhesive Silicone Paper to Facilitate easy removal.

Bite Registration Wax CE

Thickness : 1.7mm to 2.0mm (Custom Thickness Catered),

Size : 160mm X 90 mm

Packing : Each Packet of 'MAARC' Bite Registration Wax contains 10 sheets wrapped in non adhesive Silicone Paper to facilitate easy removal. Available in 250gms pack appx. Bite Wafers pack of 10

Available in : 3 variants - Hard, Extra Hard & Zero Tolerance

Packing	Product Code
Bite Registration Wax (Aluminium Filled)	2202/010
Classic BRW Extra Hard (Aluminium Filled)	2207/010
Perfect Bite (Zero tolerance)	2213/010

Properties & Advantages :

- ▶▶ Contains Aluminium particles which gives even heating
- ▶▶ Dimensionally stable, does not shrink or distort.
- ▶▶ Ideal for checking and maintaining accurate occlusal records.
- ▶▶ When heated at 55°C it turns in to 'Dead Soft' in less than 1 minute.
- ▶▶ Adapts to every details and sets by water or room temperature i.e. 27°C.

Shelf life and storage : Best before 60 months under proper storage.



Benefits :

- ▶▶ **Accurate Bite:** Even Heating Due to Aluminum Particles.
- ▶▶ **No Shrinkage:** Negligible Shrinkage Due to Crystallization Around Aluminum & Resin.
- ▶▶ **No Distortion:** Very Tough so Lesser Change of Distortion.

Green Tracing Sticks CE



Thickness : 113 x 9 mm dia weighing appx. 10gms.

Packing	Product Code
10 Sticks	1501/010
15 Sticks	1501/015
100 Sticks	1501/100

Indication :

- ▶▶ Low fusing material primarily used for border extensions on impression trays.
- ▶▶ High viscosity also makes it ideal when recording the full depth of the sulcus.
- ▶▶ Modify dentate tray by building up material where tooth has been extracted.
- ▶▶ Make functional and relining impressions.

Properties & Advantages :

- ▶▶ Suitable for muscle trimming work.
- ▶▶ Fast and uniform plasticity.
- ▶▶ Quick hardening in cold water.
- ▶▶ Wide working area.

Shelf life and storage :

Best before 60 months under proper storage.

Shellac Base Plates CE

Thickness : Upper / Lower Base Plate: 1.3mm, 1.7mm, 2.0mm, 2.5mm and 3.0mm (+/- 0.15mm)
(Custom Thickness Catered)


Colour : Brown / Pink / Aluminium

Packing	Product Code
Brown Base Plate (8 Upper + 4 Lower)	1001/084
Yellow base Plate (8 Upper + 4 Lower)	1024/084

Type : It comes in different formulations for different climatic conditions (LMP / HMP / Al. Filled)

Properties & Advantage

- ▶▶ They are ready to use Maxillary & Mandibular plates - 8 Upper & 4 Lower.
- ▶▶ Optimum heat resistance to minimize overheating of the Plate.
- ▶▶ Easily adaptable on heating and easy to work with.
- ▶▶ Retain the shape stability and do not warp back on cooling.
- ▶▶ Optimum mechanical strength to resist breaking during Try-in denture
- ▶▶ Does not adhere to plaster.

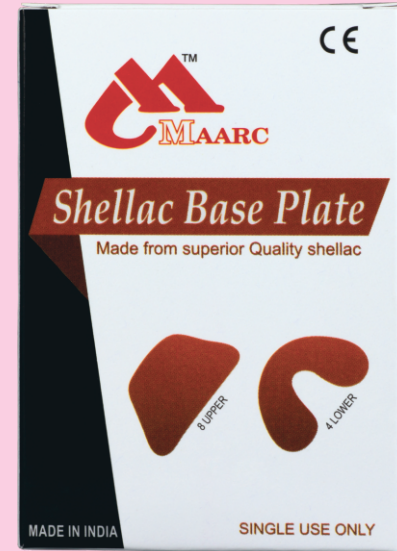

Shelf life and storage : Best before 24 months. Store below 30°C under proper storage and avoid from direct sunlight.

Impression Composition CE



Packing	Product Code
Pack of 5 cakes of appx. 200gms	1503/005

Properties & Advantages :

- ▶▶ It softens at 60°C.
- ▶▶ It is excellent for taking primary impression of edentulous teeth and becomes stable after cooling and are shrinkage free.
- ▶▶ It can be used with matrix band for single tooth impressions.

Shelf life and storage : Best before 60 months under proper storage.


Special Tray Plates CE

Packing	Product Code
10 pcs.	1309/010

Benefits Of Special Tray Plates :

- ▶▶ Time Saving and economical as compared to light cure and cold cure tray.
- ▶▶ No toxic gases as in cold cure.
- ▶▶ STP is low fusing thermo-plastic compound with high flow. So does not melt wax spacer and flows through stoppers.
- ▶▶ It has heat conducting properties, so no worries when applying hot border moulding compound. Heat spreads immediately and plate dimension remains unchanged.
- ▶▶ Stronger than conventional Shellac Plates.
- ▶▶ Easily fused to each other so handle can be made from excess.
- ▶▶ Excess can be cut easily with scissors when hot and when cold, trim with burs, abrasive wheels, etc.
- ▶▶ Borders can be folded easily.
- ▶▶ Modify the tray anytime so best border moulding can be achieved.

Shelf life and storage : Best before 24 months under proper storage.

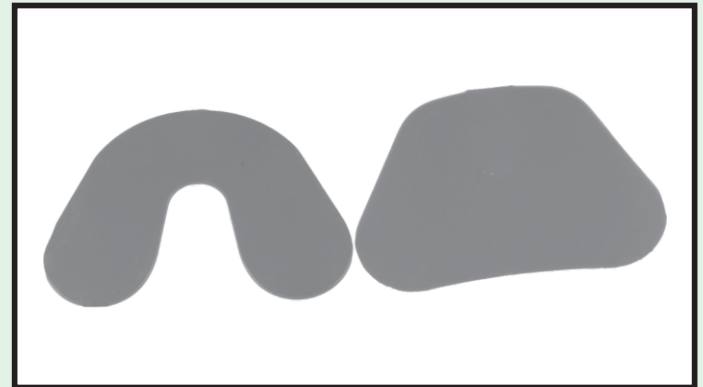

Modelling Wax / Sheet Wax / Base Plate Wax / Hard Wax CE



Thickness : 1.3mm - 1.5mm. (Custom Size / Thickness Catered),

Size : 160mm X 90 mm

Packing	Product Code
Modelling Wax Regular	(12 Sheet-2001/012)-(500 gms-2001/500)
Modelling Wax Red	(12 Sheet-2002/012)-(500 gms-2002/500)
Plasti wax Type 6	(12 Sheet-2015/012)
HARD WAX	(12 Sheet-2010/012)-(500 gms-2010/500)

Color : It comes in various shades such as pink, light pink, red.

Type : Soft, Medium, Hard & Plasti Wax.

Properties & Advantages :

- ▶▶ The wax is formable at slight increased temperature.
- ▶▶ Sufficient hard and resistant on cooling to prevent occlusion changes during trying with patient before making Final Denture to prevent failure of final denture.
- ▶▶ Easy & Clean carve on teeth.
- ▶▶ Bends without cracking.

Shelf life and storage :

Best before 60 months under proper storage.

Inlay Sticks CE



Thickness : L 114 x 8 mm dia of Big Sticks

Packing	Product Code
10 Sticks (70 gms)	2403/010

Properties & Advantages :

- ▶▶ It comes in stick form.
- ▶▶ It is used in institutes by students for easy application.

Shelf life and storage :

Best before 60 months under proper storage.

Casting Wax Sheets CE



Thickness : 0.3 to 0.6 mm

Size : 160mm x 90 mm

Colour : 'Maarc' Casting Wax sheets are thin wax sheets. Available in Green / Blue color for cobalt chrome cast partial dentures.

Packing : Available in 200gms pack. The Sheets are wrapped in non adhesive Silicone Paper to Facilitate easy removal.

Properties & Advantages :

- ▶▶ The Sheets are sufficiently thin to have thinner casting as possible without defects to reduce machining.
- ▶▶ They easily burn out during castings.

Shelf life and storage :

Best before 60 months under proper storage.

Die Stone Plaster (Pink) CE



For crown and bridge, dies and cast partials. For basing dental arches and opposing models. For making study models. In use where economical yet tough, non brittle and durable models are required.

Packing	Product Code
30 kg	4101/030

Type : IV - Specification :

Water/ Powder	24-26ml/100gm	Hard Set (Initial)	30 Minutes
MixingTime	45 seconds wet	Wet Compressive Strength	400 kg/cm2
WorkingTime	5 minutes dry	Dry Compressive Strength	800 kg/cm2
SettingTime	8-12 minutes	Linear Expansion (Max)	0.08%

Shelf life and storage :

Best before 24 months when stored in cool and dry place.

Reinforcement Mesh CE



Thickness : 0.25 mm

Type : Golden and Silver

Packing	Product Code
Golden Pack of 10 Pcs	5502 / 010
Silver Pack of 10 Pcs	5501 / 010

Properties & Advantages :

- ▶▶ Re-inforcement Mesh is made from 316 grade stainless steel plated with gold or plain polished .
- ▶▶ These Mesh are very much flexible and can easily adapt any shape.
- ▶▶ It is 0.25 mm thin so can be easily layered between acrylic.
- ▶▶ It gives denture desirable strength.

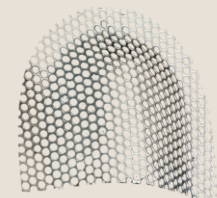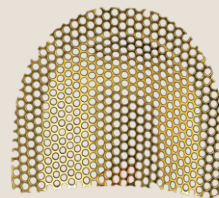

Cold Cure / Rapid Repair Powder & Liquid



Packing	Product Code
Cold Cure Powder Clear	(100 gms-6201/100)-(400 gms-6201/400)
Cold Cure Powder Pink	(100 gms-6202/100)-(400 gms-6202/400)
Cold Cure Powder Light Veined	(100 gms-6203/100)-(400 gms-6203/400)
Cold Cure Liquid	(100 ml-6301/100)-(400 ml-6301/400)
Cold Cure Universal Pack Clear	400 gms + 400 ml(6504/001)
Cold Cure Universal Pack Pink	400 gms + 400 ml(6505/001)
Cold Cure Universal Pack Light Veined	400 gms + 400 ml(6506/001)



Properties & Advantages :

- ▶▶ This is a 2 part rigid dental acrylic used for making dental appliances.
- ▶▶ Contains both powder and liquid components.
- ▶▶ Pink Dental Acrylic stimulates the fleshy gum tissue of the mouth.
- ▶▶ By combining this with pre-made acrylic teeth, or with your own castings made from tooth shade acrylic can make or repair dentures.
- ▶▶ For best results use with MAARC cold cure liquid.

Description & Features:

- ▶▶ Working time of 4 minutes
- ▶▶ Rapid bench cure time of 14 minutes
- ▶▶ Also useful in orthodontic work

Indication for use : Used for full or partial upper or lower dentures.

Shelf life and storage : Best before 24 months under proper storage.

Cold Mould Seal / Separating Medium

Packing	Product Code
110 ml	6401/110
5 Liter	6401/005

Properties & Advantages :

- ▶▶ Cold Mould Seal is a specially developed formulation of separating solution for universal use, suitable for both packing / pressing technique & cast moulding.
- ▶▶ Makes tough film with elasticity which is unbreakable in pressure.
- ▶▶ Film does not chip off from plaster at pressure. High quality of the raw material guarantees optimum separation with both techniques.

Shelf life and storage : Best before 24 months under proper storage.



Heat Cure / Cross Linked Powder & Liquid ☺



Packing	Product Code
Heat Cure Powder Clear	(100 gms-6001/100)-(450 gms-6001/450)
Heat Cure Powder Pink	(100 gms-6002/100)-(450 gms-6002/450)
Heat Cure Powder Light Veined	(100 gms-6003/100)-(450 gms-6003/450)
Heat Cure Liquid	(100 ml-6101/100)-(225 ml-6101/225)
Heat Cure Universal Pack Clear	450 gms + 225 ml + 110 ml CMSL(6501/001)
Heat Cure Universal Pack Pink	450 gms + 225 ml + 110 ml CMSL(6502/001)
Heat Cure Universal Pack Light Veined	450 gms + 225 ml + 110 ml CMSL(6503/001)



Properties & Advantages :

- ▶▶ This is a 2 part rigid dental acrylic used for making dental appliances.
 - ▶▶ Contains both powder and liquid components.
 - ▶▶ Light Veined Pink Dental Acrylic stimulates the fleshy gum tissue of the mouth.
- Indication for use : Used for full or partial upper or lower dentures

Description & Features

- ▶▶ Heat cured acrylic denture base material.
- ▶▶ Simple mixing, easy to process.
- ▶▶ Cross linking ensures superior quality.

Shelf life and storage : Best before 24 months under proper storage.
For Best results, Use with MAARC Heat Cure Liquid.

Description & Features:

- ▶▶ Working time of 4 minutes
- ▶▶ Rapid bench cure time of 14 minutes
- ▶▶ Also useful in orthodontic work

Sticky Wax Sticks ☺

Thickness : 114 x 8 mm dia sticks weighing appx. 6gms. / Pc

Packing	Product Code
10 pcs	2702/010
200 gms	2702/200

Properties & Advantages :

- ▶▶ It is blending of various waxes and resins and it is yellow in color.
- ▶▶ It melts when heated and adheres to the surface over which it is applied.
- ▶▶ It holds the components prior to fixation and gives a clean burnout.

Shelf life and storage : Best before 60 months under proper storage.



ETC-37 CE



Known as : Etching Gel - 37% Ortho Phosphoric Acid

Packing	Product Code
3 gm syringe + 3 Applicator tips	9401 / 003
25 gm syringe + 10 Applicator tips	9401 / 025
15 ml Liquid	9402 / 015

Properties & Advantages:

- ▶▶ It is used in total etch technique for composite restorative, adhesives cementation of inlays, crowns, bridges, veneers, adhesive bridges and posts & cores.

Benefits :

- ▶▶ Controlled Flow
- ▶▶ Precise Placement
- ▶▶ Selective Etching
- ▶▶ High Visibility
- ▶▶ Higher Resin Penetration, Hence Better Mechanical Retention

Shelf life and storage :

Best before 24 months under proper storage.

Description :

- ▶▶ Thixotropic gel
- ▶▶ Silicon free

Eugenol

Packing	Product Code
30 ml	3002 / 030
110 ml	3002 / 110

Properties & Advantages :

- ▶▶ 98.9% pure.
- ▶▶ For temporary relief of localized dental pain when combine with zinc oxide to form a sedative dressing.
- ▶▶ Chemically pure with increased effectiveness.
- ▶▶ Forms a uniform paste consistency when mixed with zinc oxide.
- ▶▶ Forms a sedative dressing or lining under non composite resin when mixed with zinc oxide.

Shelf life and storage : Best before 24 months under proper storage.



Calcium Hydroxide Powder

Indication :

- ▶▶ Ideal for temporary root filling
- ▶▶ Pulp capping
- ▶▶ Pulp protection
- ▶▶ Provides insulation in deep cavities

Composition :

- ▶▶ Chemically pure calcium hydroxide in powder form

Properties & Advantages :

- ▶▶ Provides bactericidal effect on carious dentine.
- ▶▶ Paste delivered to root apex provides pulp insulation.
- ▶▶ Protects the vital pulp from infections and further decay.
- ▶▶ Can be easily removed from root canal by simple irrigation

Shelf life and storage :

Best before 24 months under proper storage.

Packing	Product Code
15 gms	5002 / 015



Restorative Range

Mercury



Packing	Product Code
30 gms	3001 / 030
225 gms	3001 / 225

Properties & Advantages :

- ▶▶ 99.99% Pure
- ▶▶ An especially triple distilled, chemically pure mercury.
- ▶▶ Recommended for preparing superior dental amalgam in combination with either of the Alloys in powder form.

Zinc Oxide



Packing	Product Code
100 gms	5001 / 100

Properties & Advantages :

- ▶▶ 99.99% Pure
- ▶▶ Arsenic free, pure zinc oxide powder for temporary dental dressing.
- ▶▶ Used for restorative and prosthodontic applications.
- ▶▶ For best result use with MAARC Eugenol

Shelf life and storage : Best before 60 months when stored in cool and dry place.

Lubri Spray



Packing	Product Code
650 ml Can + Nozzle.	5506 / 650

All purpose Hand piece Lubricant Spray for maintenance of all dental hand pieces including highspeed, slow speed and air motors.

Indications: Highly effective lubrication for all high and slow speed dental rotary hand pieces. Lubri Spray contains anti-rust agent that prevents micro rust on inner bearings. High penetrative ability to ensure thorough coating of bearing surface. Each can contains a high volume pressure to provide maximum bearing rotation during oiling. High internal pressure of 70+ Psi (4.8atm) at 20°C. Cans are compatible with the most renowned brands. Clear in colour.

Directions for use: Insert into back of hand piece and spray through for one to two seconds immediately after each use and before sterilization. Wipe excess thoroughly. Important: Hold can in a vertical position when in use. Refer to hand piece maintenance instructions where applicable.

Shelf life and storage :

Best before 36 months under proper storage.

Mouth Mirrors

These are Anti-Fogging Mouth Mirrors. They are so hard that they don't get scratched easily. They are unbreakable and when it comes to reflection, in "MAARC" mirror its bright, sharp and no double reflection like cheap, quality mirrors. It can be sterilized (Autoclavable). Even through its expensive, overall, life span and reflection quality makes it competitive to regular mirrors.



Arch Bar CE



Packing	Product Code
Arch Bar 12 inch long	5508 / 012

Description: It is made of 316 grade SS suitable for rigid fixation, techniques in the dentate patient begin with fixation of the occlusion. This ensures that the patients maintain their preoperative occlusal status. There are several techniques to providing mandibulomaxillary fixation (MMF). Many surgeons agree that the gold standards in MMF is, the use of arch bars.

Indications: For temporary fragment stabilization in emergency cases before definitive treatment.

As a tension band in combination with rigid internal fixation,

- ▶▶ For long-term fixation in conservative treatment
- ▶▶ For fixation of avulsed teeth and alveolar crest fractures

Glass Beads CE



A method used for sterilizing the working ends of endodontic files and reamers or small dental instrument tips by placing them in a container containing glass beads heated to approximately 225°C (437°F) for a defined period of time.

Packing	Product Code
Glass Beads 200 gms	5503 / 200

Relief Wax for Ortho CE



Packing	Product Code
(5 Strips)	2310 / 005

Properties & Advantages :

- ▶▶ Relief Wax consist of a medical grade silicone for temporary relief from appliance irritation.
- ▶▶ This patient relief wax lasts longer and is easier to use than ordinary orthodontic waxes.
- ▶▶ It is clear and waterproof and stays on the bracket longer for greater patient relief.

Shelf life and storage :

Best before 24 months under proper storage.

EXHIBITIONS 2019

DATES	EXHIBITIONS	STALL NO.
4 - 6 January	Kerala State Dental Conference	B-25
18 - 20 January	IDC Indore	09
5 - 6 - 7 February	AEEDC-Dubai	TO BE ANNOUNCED
16 - 17 February	EXPODENT KOLKATA 2019	G-28-29
16 - 17 February	FAMDENT SHOW - PUNE 2019	TO BE ANNOUNCED
16 - 17 February	Gujarat Dental Show 2019	27-36
12 - 16 March	International Dental Show Germany - 2019	F-80 / G-81
23 - 24 March	Guwahati dental show 2019	4
30 - 31 March	EXPODENT CHENNAI 2019	A-50 - A-51 - A-52
13 - 14 April	International Dental Lab Expo & Conference 2019 in Hyderabad	TO BE ANNOUNCED
4 - 5 May	Dental Mahotsav Lucknow-2019	A-16
7 - 8 - 9 June	FAMDENT SHOW - MUMBAI 2019	TO BE ANNOUNCED
7 - 8 September	FAMDENT SHOW - Hyderabad 2019	TO BE ANNOUNCED
Coming Soon	EXPODENT MUMBAI 2019	TO BE ANNOUNCED
20 - 21 - 22 December	EXPODENT INTERNATIONAL INDIA 2019	TO BE ANNOUNCED



Shiva Products

ISO 9001 : 2015 & 13485 : 2016

 **Manufactured by :**
Shiva Products
H/O : 214, Blue Bell Indl. Estate, Bilalpada,
Chinchpada, Vasai (East)
Dist - Palghar - 401 208 Maharashtra, India
Mobile : +91-9987637290

Branch Office
Shop No. 7, Gola Market,
Pratap Street,
Nr. Golcha Cinema, Darya Ganj,
New Dehli-110 002. (INDIA)

E-Mail : info@maarc dental.com



Web : www.maarc dental.com